



S-Mono




S-Mono

Mini Implant

- Adequate for narrow upper and lower anterior
- One-piece design with high strength for bite force
- RBM surface for enhancing osseointegration

S-Mono Lineup

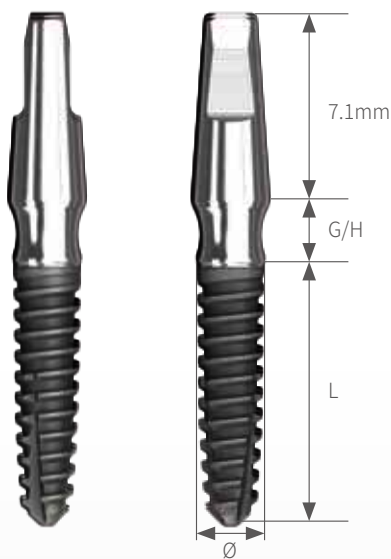

Length (mm)	G/H (mm)	ø2.5	ø3.0
8.5	2.5	MB2508SR	MB3008SR
10		MB2510SR	MB3010SR
11.5		MB2511SR	MB3011SR
13		MB2513SR	MB3013SR
8.5	4	MB2508LR	MB3008LR
10		MB2510LR	MB3010LR
11.5		MB2511LR	MB3011LR
13		MB2513LR	MB3013LR

Impression Coping
Use for taking impression

MBICP



Temporary Cap
Use for temporary prosthesis

MBTCP



Lab Analog
Use for dental laboratory

MBAP



S-Mono Surgical Kit



S-Mono Kit(MBKT)

Drill Extension



SDE00

Point Drill



MBPD15S

Ø1.8 Drill Short



MBSD18S

Ø1.8 Drill Long



MBSD18L

Ø2.3 Drill Short



MBSD23S

Ø2.3 Drill Long



MBSD23L

Direction Indicator



MBDI00

Fixture Driver T/W



TWFD00

Fixture Driver H/P



HFDP00


Torque Wrench
(Option)



TW00

S-Mono Surgical Procedure

- **Drilling Speed** : Recommend 1200~1500rpm
- **Insertion Speed** : Recommend 20~40rpm
- **Insertion Torque** : Recommend under 45Ncm

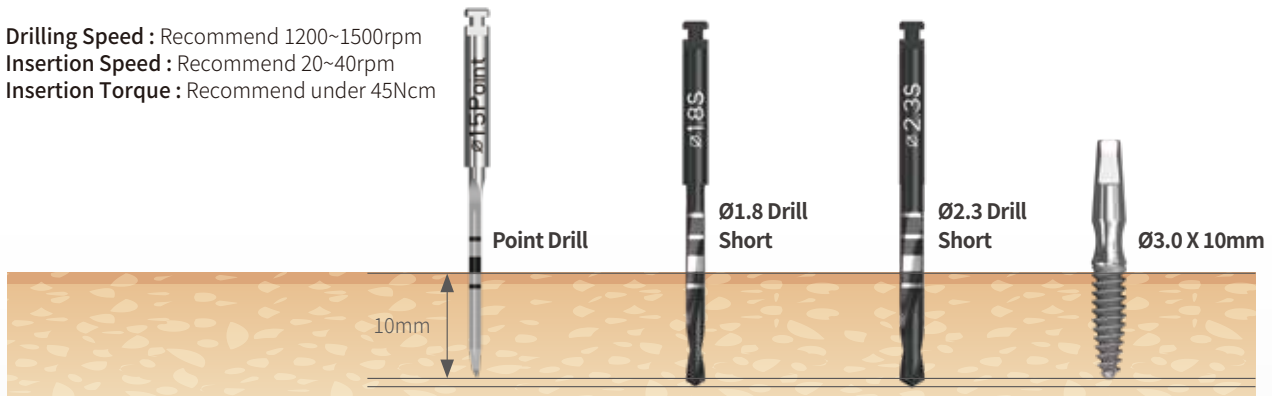

	Bone Quality	Point Drill	Ø1.8 Drill	Ø2.3 Drill	Fixture
Ø2.5	Soft	○			
	Medium	○	○		
	Hard	○	○		
Ø3.0	Soft	○	○		
	Medium	○	○	○	
	Hard	○	○	○	

S-Mono

SIS-DENTAL INC

6131 Orangethorpe Ave Suite 410, Buena Park, CA 90620, USA
Tel : 562-550-0001
E-mail : info@sis-dental.com
www.sis-dental.com